

A SIMPLE CHANGE FOR THE BETTER



Medtronic

MINIMED™ MIO™ ADVANCE SET CHANGES MADE EASY

FEWER STEPS, FASTER INSERTION

Fill your reservoir as per your pump instructions.

Prime your infusion set by filling the tubing until you see drops appear.



PEEL ADHESIVE



PLACE ON BODY



PINCH AND REMOVE SAFETY PIN



PRESS THE BUTTON



MINIMED™ MIO™ ADVANCE requires a cannula fill of **0.6 units**

* For quick reference only. For full instructions please refer to the IFU.

For a training video and further details please visit:
www.medtronic-diabetes.co.uk

United Kingdom

Medtronic Limited
Building 9, Croxley Park
Hatters Lane,
Watford, Herts WD18 8WW
Tel: +44 (0)1923 205 167
Fax: +44 (0)1923 241 004
www.medtronic-diabetes.co.uk

Medtronic Ireland Limited

Block 3090-3094,
Lake Drive
Citywest Business Campus,
Dublin, DN24 XN47
Tel: +353 (0)15111499
Fax: +353 (0)18077220
www.medtronic-diabetes.ie

